

PROLYTE ANALYZER

2. b Component Installation and Replacement

Installation and replacement of all consumable components will be outlined in the following sections. Please refer to this section when replacing any components following the maintenance schedule outlined below.

RECOMMENDED MAINTENANCE/REPLACEMENT SCHEDULE

| | | Replacement PN |
|---------------------------|---|-----------------------|
| Membrane Assembly | Guaranteed Use Life 3 Months | ME-2258D |
| Sample Tube | Guaranteed Use Life 3 Months | ME-2104D |
| Pump Tubing | Guaranteed Use Life 3 Months | ME-2104D |
| Internal Filling Solution | Guaranteed Use Life 3 Months | ME-2492D |
| Solutions Valve | Guaranteed Use Life 12 Months | ME-2108D |
| Sample Probe | Guaranteed Use Life 12 Months | ME-2107D |
| Sample Detector | Replace as Necessary (see Diagnostics) | ME-2257D |
| K+ Electrode | Replace as Necessary - Guaranteed Use Life 3 Months | ME-2101D |
| Na+ Electrode | Replace as Necessary - Guaranteed Use Life 6 Months | ME-2102D |
| Cl- Electrode | Replace as Necessary - Guaranteed Use Life 3 Months | ME-2113D |
| Reference Electrode | Replace as Necessary - Guaranteed Use Life 12 Months | ME-2103D |
| Solutions Pack | Change as Necessary - Use Before Install by Date | IL-2121D |
| Probe Wiper | Replace after 100 Serum/Plasma Samples, 50 Whole Blood Samples, or 2 weeks, whichever comes first | ME-2323D |
| Printer Paper | Replace as Necessary | ME-2541D |